


Zoom-Ex / Zoom-Office

Training Guide and User Manual OS 64 bit Windows 8 and Windows 7

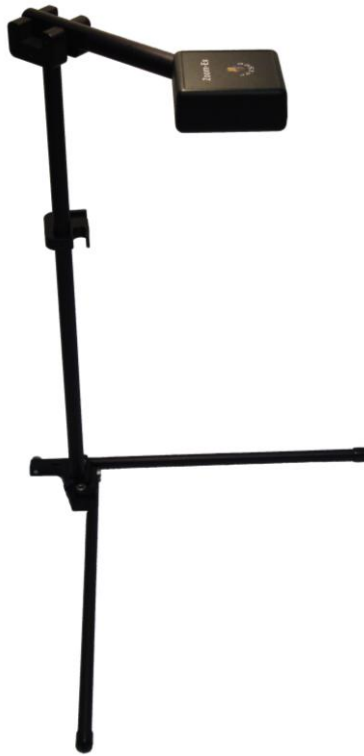

Version 5.1

**Freedom Scientific, Inc.
11800 31st Court North
St. Petersburg, FL 33716-1805**

**(727) 803-8600
Support@FreedomScientific.com
www.FreedomScientific.com**

Table of Contents

What are Zoom-Ex and Zoom-Office?.....	3
Setting Up Zoom-Ex.....	3
How to Place a Document Under the Camera	4
Keyboard Keys to Navigate Reading	4
Viewing and Reading Magnified Graphics.....	5
Changing Languages.....	5
Local CCTV	6
Working in Formatted Text Mode	7
Various Formatted Modes to Follow Text on the Screen	7
Other Formatted Mode Options	8
Navigate Text in Full Screen Formatted Mode.....	9
Switching between Formatted Mode and CCTV Mode.....	9
Going to the Next Page or another Document	10
To Save and open a Document as a Text File	11
To view and read document saved on your computer in PDF format	12
To Save a Document as an MP3 (Audio) File.....	12
Opening an Audio File.....	13
Printing a Document	13
Changing the Display Characteristics	13
Rotating the View of an Object.....	14
Using the Preview Window	14
Changing the Advanced Display Characteristics	15
Creating and Managing Books.....	16
More Tips on Saving a Book	20
Reading a Saved Book	20
Managing Books	22
Useful Tips for the Zoom-Ex Camera	25
Zoom-Ex Quick Reference Guide (5.0)	26

What are Zoom-Ex and Zoom-Office?

Zoom-Ex is a computer-connected portable electronic scanner/reader and magnifier that helps those with low vision to view, read and magnify small print.

Zoom-Ex reads aloud books, magazines, newspapers, documents, prescription labels and other print in a variety of naturally-sounding voices and languages.

Zoom-Office is a software program necessary to operate Zoom-Ex. It is fully customizable to accommodate individual preferences. Default settings can be changed by going to **Applications** and down to **Settings**.

Setting Up Zoom-Ex

2.1 Remove the device from the box and unfold the two legs. Pivot the two pins at the base of the vertical camera stand outward over each leg. This locks the legs in position (Picture 1).

2.2 Now, this next part is very important. Place the software installation CD into your computer and follow the installation instructions. Do not plug the USB cable into your PC until the installation tells you to do so.

2.3 When instructed by the setup CD, plug the USB cable directly into the USB2 port on your computer or laptop. Always plug the USB cable directly to your PC. Do not use bus-powered USB hub.

Now Zoom-Ex is ready to work. To locate the Zoom-Office application, press the **Windows** key plus the **D** key to open the desktop. Once on the desktop, press the **Z** key to locate Zoom-Office. When you locate the application, open it. Once Zoom-Office opens up, it will announce "Camera Started." This indicates that the camera is ready to start scanning.

Zoom-Ex should be in the standing position. It has two horizontal legs: the longer leg is 11 ½ inches while the shorter leg is 9 inches long. These two legs create a corner guide to assist a user in properly aligning documents to be scanned.

Two modes of operation are available: Scanning and CCTV Local (Local Live Magnification).

Scanning Mode: When Zoom-Office starts up, the Zoom-Ex camera is turned on and is ready to scan. It takes a snapshot of the document placed under the camera, creates an image of the page and displays it on the screen. Zoom-Ex is then ready to read the text aloud.

CCTV Local Mode (Close view - Live Magnification Mode): Zoom-Ex displays what is currently placed under its camera on the computer screen. In this mode you can fill out forms or view small objects or pictures under the camera. You can toggle between Scanning and Magnification mode by pressing the **Y** key on a computer keyboard.

Quick Start

How to Place a Document Under the Camera

Place a document under the Zoom-Ex camera, press the space bar to hear: "Camera started"

1. To scan and hear printed material read aloud, press **E** on the keyboard, and wait until you hear a shutter sound. Reading starts immediately.
2. Press **X** to pause and resume reading.
3. To preview the document press **V** or **F5**. Press **+** or **-** to Zoom In or Zoom Out.
4. To view the document in the **Formatted Text Mode**, Press the **F** key or **F2**. Press **+** to increase the font size. Press **-** to decrease the font size.
5. Press **E** to have the text read to you from the beginning of the document. Follow along as the text is read aloud and displayed on the screen.

Keyboard Keys to Navigate Reading

1. Place printed material under the camera.
2. Press the **Space Bar** to turn the camera on. You will hear "Camera Started"
3. Press the **E** key on a computer keyboard to start reading from the beginning of the page. You can also start reading by going to the Menu's **Voice** Tab and choosing **Speak Entire Page**.
4. Press the **X** key to stop or resume reading.
5. Press the function **F2** or **F** key on your keyboard to view the formatted text and read on the screen what is being read aloud.
6. Press **Shift + E** to hear the page layout description.
 - a. Number of words on the page
 - b. Number of sections on the page
 - c. Percentage of text on the page
 - d. Number of columns and headers on the page
7. Press **Control + Right Arrow** to move to the next word.
8. Press **Control + Left Arrow** to move to the previous word.
9. Press **Down Arrow** to move to the next sentence.
10. Press **Up Arrow** to move to the previous sentence.
11. Press **Control + Down Arrow** to move to the next section
12. Press **Control + Up Arrow** to move to the previous section.
13. Press **Right Arrow** to move to the next character
14. Press **Left Arrow** to move to the previous character


To start reading at an exact point, place your cursor on a word and press the mouse wheel. Zoom-Ex will start reading at the word your cursor is on. To stop the document being read, press the **Control** key.

Viewing and Reading Magnified Graphics

Text, which is currently read, does not have to be seen on the screen. You can move the image on the screen to view a picture, graph, etc. and listen to the voice reading the description of the page at the same time. If there is only one picture on the page, move the picture to the center of the screen, magnify the picture (see **CCTV Mode**) and click the **E** key.

Changing the Settings


You can change the voice parameters by going to **Application, Settings** then **Voice Tab**:

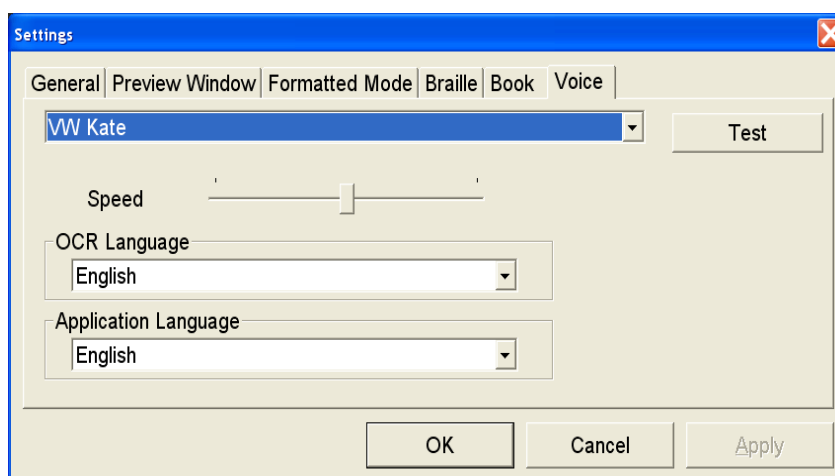

Test different voices, select the one that you like and click Apply. Change the speed of reading using the slide bar or by pressing **Shift + Q** to make it faster or the **Shift + Z** to make it slower.

Changing Languages

Go to **Application** and down to **Settings** and over to the **Voice** tab. You will see that there are two other options that you can modify. The first is **OCR Language**.

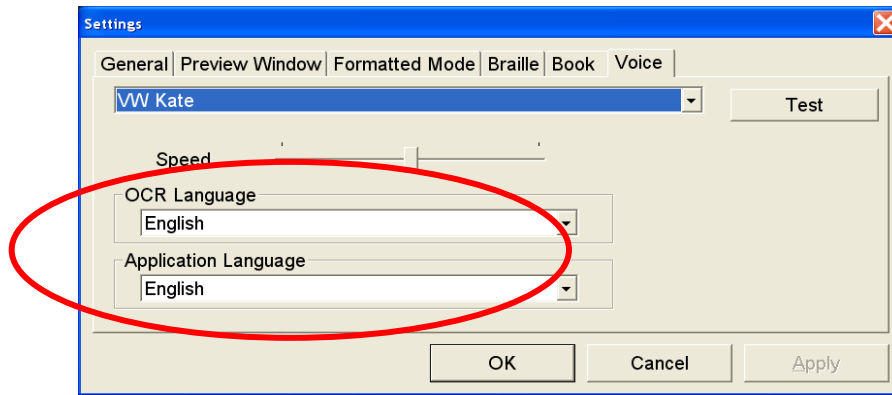


OCR Language is the language of the document that you are putting under the Zoom-Ex camera. By default, the **OCR Language** is English.

The second option is **Application Language**. It is the language of the application. This includes text, menus, and dialog boxes. By default the **Application Language** is English.

Local CCTV

Go to **Applications** on the tool bar, go down to **Settings** and choose “**Use arrow keys to simulate X-Y table**” under the **General** Tab.

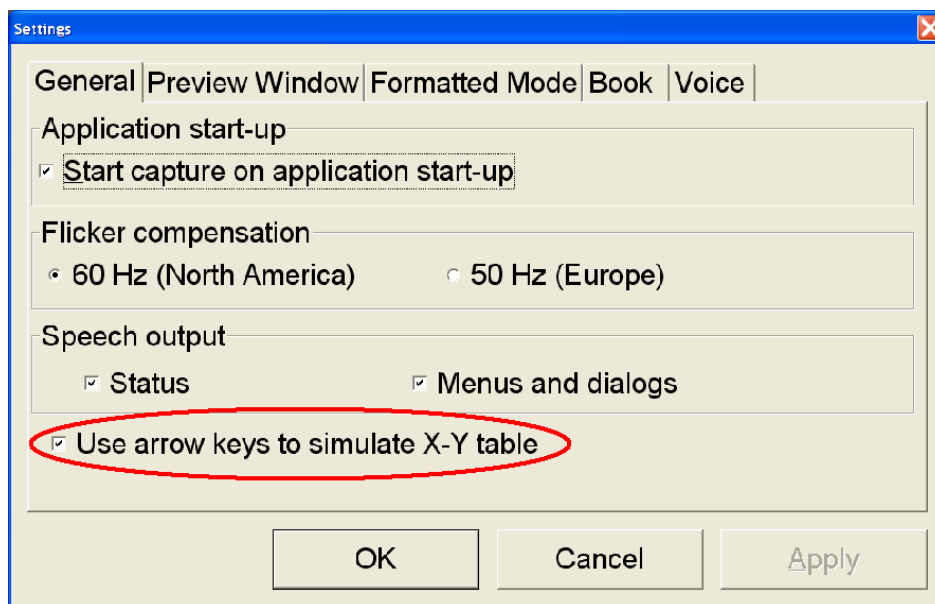


If this box is checked, arrow keys will behave as described below, meaning you can navigate the screen in **CCTV Mode** with arrow keys. Use the **Up** and **Down Arrow keys** to scroll up and down in both the CCTV and **Formatted Mode**.

If this box is unchecked the arrow keys can be used for reading functions as described in section “Notebook (Laptop) keystrokes for reading functions”. Default value for this setting is unchecked.

Take a picture, then press **V** to preview the image.
Navigate through the image by using arrow keys.
Press "+" and "-" to zoom in or out on the image on the screen.
Alternatively, Roll the mouse wheel forward/backwards to zoom in and out.
Press **E** to hear text read loud while you are navigating through the image

Working in Formatted Text Mode

Formatted Text Mode separates text from pictures and wraps the text according to the selected magnification. Pictures are "ignored" and Zoom-Ex focuses just on the text. A document is scanned and changed over to true font, as in a word document. The text is displayed in one column and no matter how much you magnify it, the text will not run off the screen. There is no need for an X-Y table.

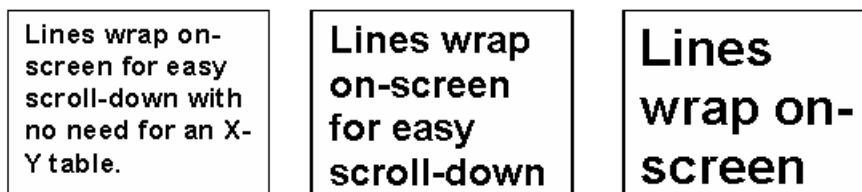

Various Formatted Modes to Follow Text on the Screen

Go to **Application** and down to **Settings**, click on the tab labeled **Formatted Mode**, and go down to the section called **Formatted Mode**. There are four options to read formatted text: **Full Screen**, **Highlighted Line**, **Auto Scroll**, and **Word by Word**.

The first option, **Full Screen**, is the DEFAULT. It will display text on the screen as if it was a word document. The **Full Screen** option highlights the word as it is being read.

The next option is **Highlighted Line**. This option has one line that goes across the screen and it highlights the word as it is being read. This allows you to follow along with the reading.

The next option is **Auto Scroll**. Auto scroll is an option that keeps the text moving in a running line, at a constant speed across the computer screen.


Auto scroll is controlled by using the (<) and (>) keys. (<) slows down the scrolling text line, while (>) increases the scrolling text line. Auto scroll is used to read the flowing words across the computer screen, with no highlighted bar or spoken words. To stop or pause the auto scrolling, press the **X** key. To resume or start auto scrolling press **X** or the (>) key.

Note: You cannot use speech output with auto scroll.

The last option is **word by word**. The **word by word** option takes one word at a time and shows it on the screen as it is simultaneously being read.

You have the ability to change the size of the font seen on the screen by pressing “+” to increase the font and “-” to decrease the size of the font.

You can also change the speed at which the words are being spoken by pressing (**Shift + Q**) to increase the voice speed or (**Shift + Z**) to decrease the voice speed.

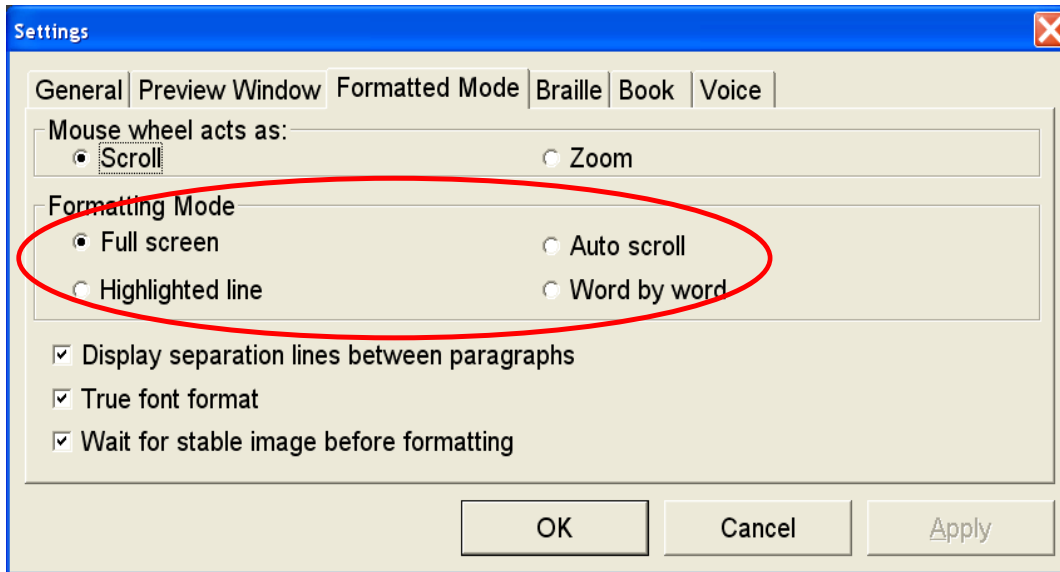

Other Formatted Mode Options


You also have the option to turn on/off the **Display separation lines between paragraphs** as well as **True font format**. The option **Display separation lines between paragraphs** puts lines on the screen separating paragraphs. If disabled, the text will flow the same as a word document.

If **True font format** is enabled, the text will show up on the screen as in a word document. If this option is disabled the “text” or “font” on the screen will show up just as it is in the actual document placed under the camera.

Formatted Mode without Speech Output

By keyboard: press F2 (F) key

By mouse: double-click anywhere in the text

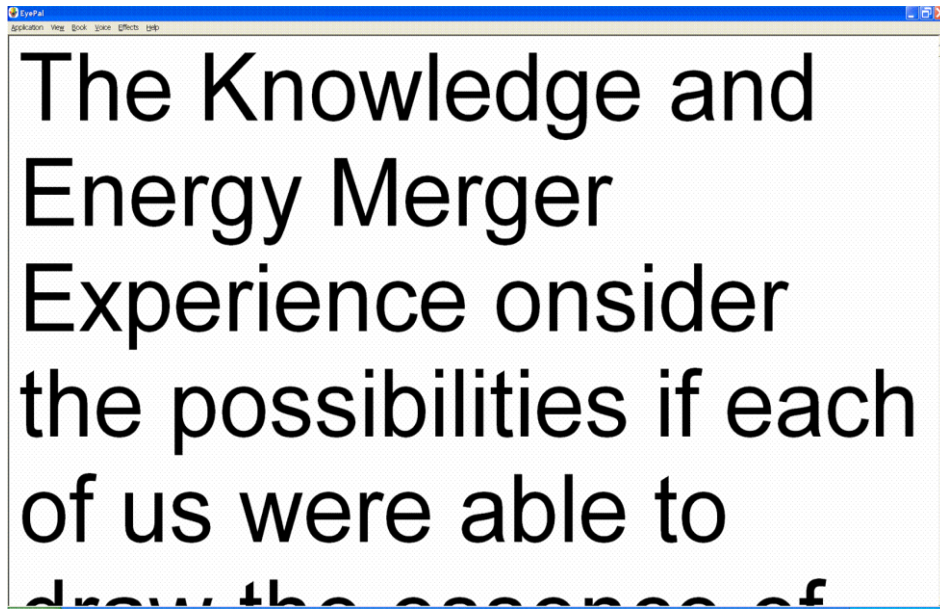

Formatting the text may take a few seconds depending on the amount of text on the page and the power of your computer.

Navigate Text in Full Screen Formatted Mode

By keyboard:

Use **Up** and **Down** arrow keys to scroll up and down line by line. Use the **Page-Up** and **Page-Down** keys to scroll up and down page by page.

Use the **Home/End** keys to go to the beginning/end of the document.

Note: This will only work if "Use arrow keys to simulate X-Y table" is checked under Settings.

Switching between Formatted Mode and CCTV Mode

To go back to Local CCTV Mode:

By keyboard: press function key F2 or F key.

By mouse: double-click anywhere in the text.

Note 1: After you switch to **Local (CCTV) Mode** from **Formatted Mode**, Press F4 (S) key in order to adjust the sharpness.

Note 2: After switching to **Local (CCTV) Mode** from **Formatted Mode**, the camera remains off, and the whole page is in the computer memory.

Originally, when you switch from **Local (CCTV) Mode** to **Formatted Text Mode**, Zoom-Ex will take a snapshot of the page and shut the camera off, in order to format the text. Once you leave **Formatted Text Mode** and return to **Local (CCTV) Mode**,

with the camera remaining off, Zoom-Ex displays the still image of the document. It makes no difference as far as reading the document is concerned. However, in order to go to a different document, you have to turn the camera back on. For instructions on how to turn the Zoom-Ex camera on, see **Going To The Next Page or Another Document** topic below.

Going to the Next Page or another Document

If the camera is on, all you have to do is place the new page at the Zoom-Ex base: it will instantly appear magnified on the computer screen if you are working in live **Local (CCTV) Mode** or in **Scanning Mode**, you will see a full image of the page that is to be scanned. Press 'Y' to toggle between these two modes.


If you are working in **Formatted Text Mode** the Zoom-Ex camera is off, **the camera has to be turned on**. To turn on the Zoom-Ex camera press the **Space Bar**.

Saving a Document and Opening a Previously Saved Document

Zoom-Ex allows you to **save** documents in **JPG, TXT and MP3** formats.
Zoom-Ex allows you to **read documents** saved on your computer in **JPG, TXT, PDF**

Place a new page at the Zoom-Ex base.

Go to **Application** on the tool bar and down to **Save Image**. A **Save Image** dialog box will appear.

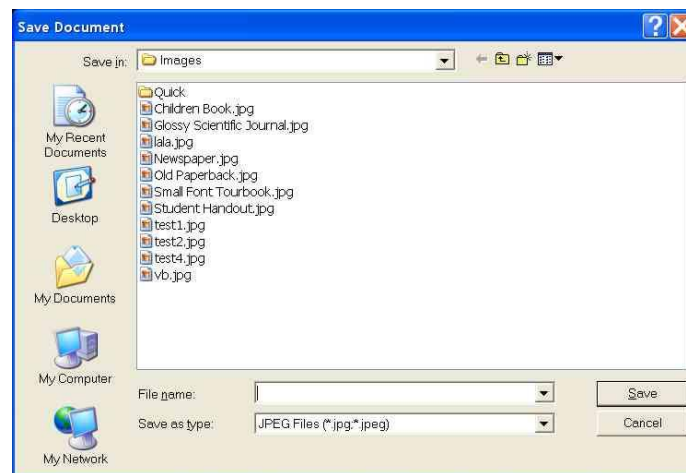


Type in the name for your document, and click the **Save** button. The image will be saved in the default directory **My Documents\Zoom-Office\Images**. You can also select another directory in which to save the document.

You can save images while in either CCTV or **Formatted Mode**. Either mode will save the image of the document.

To view a previously saved image:

Go to Application on the toolbar and go down to **Import File**.

A **Load Document** dialog box will appear.

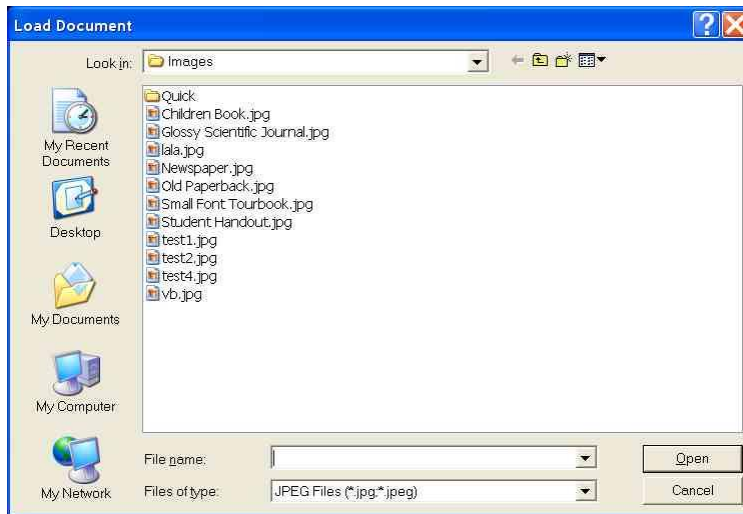

Select the image and right click on **Open**. The image will appear on the screen in **CCTV Mode**, with the camera turned off. You can manipulate the loaded image exactly as you would the real document, i.e. Format, Read, Magnify, etc.


Note: Zoom-Ex does not have to be plugged in to read previously saved images.

To Save and open a Document as a Text File

To save a document as text that is currently under the Zoom-Ex camera, press **E** to stop the camera. Once the camera is stopped you can save the document as text.

Note: Do not use SPACE BAR to stop camera unless you are able to see a stable image on the screen.

You can save text by going to the **Application** menu and down to **Save text** then press **Enter**.

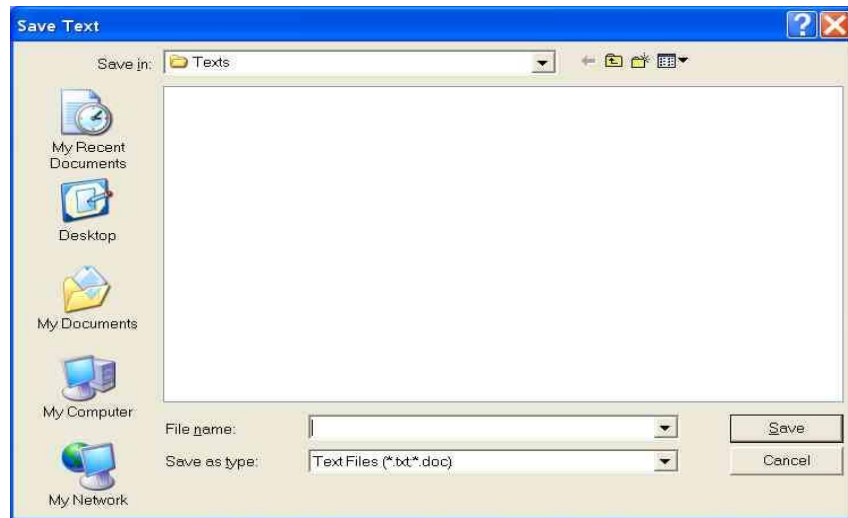

A **Save text** dialog box will appear. Type in the name for your document, and click the **Save** button. Documents will be saved in the default directory

You can also select another directory in which to save the document.

When saving a “previously saved” image as text there is no need to stop the camera because it is already stopped. Follow the same steps listed above.

To view and read previously saved as text document:

Go to Application on the toolbar and go down to **Import File**.

A **Load Document** dialog box will appear. Select a document you have previously saved in the default directory: **My Documents\Zoom-Office\Texts**.

To view and read document saved on your computer in PDF format

Zoom-Ex will not allow you to save a document in PDF format, however it allows you to open a PDF file already saved on your computer. Go to Application on the toolbar and go down to Import File. Find a directory where you have saved the file.

Note: There is not a default directory as there is for the TXT and JPG files. Find a PDF file you would like to open and read, and select it. The document will be displayed on the screen. You can read it like it was described above for the JPG and TXT documents.

To Save a Document as an MP3 (Audio) File

To save a single page document as an audio file, take a picture of this document first. Once the camera is stopped you can save the document as an audio file.

Go to **Application** menu and select **Save Audio** then press **Enter**.

A **Save audio** dialog box will appear. Type in the name for your document, and click the **Save** button. The document will be saved in the default directory **MyDocuments\Zoom-Office\Audio**. You can also select another directory in which to save the document.

Please note that when you are saving the entire book as an MP3 file (Book/Save book MP3), it will be saved in the default directory: **MyDocuments\Zoom-Office\MP3**

Opening an Audio File

An MP3 or audio file **CANNOT BE OPENED** in Zoom-Ex. Go to the directory where the file was saved and open it from there. The default directory that the MP3 files are saved in is **My Documents\Zoom-Office\Audio**. Double click on the saved file and it will open in a program such as Windows Media Player.

The saved Audio file allows you to hear the full text without having to first change it to a text format to be read. Use the audio file, burn it onto a CD and upload it to an MP3 player so you can listen to your own audio book.

Printing a Document

Zoom-Ex allows you to print out the formatted document with any magnification. Use the **F** key or **F2** key to view and print the document in **Formatted Mode**.

To preview the printed document, go to **Application** on the tool bar and go down to **Print Preview**. Print the document or go back to the formatted document and adjust the magnification.

Note: Multiple pages may be printed depending on the magnification of one page. Pages will be able to be read as a book.

Changing the Display Characteristics

Changing the Sharpness Settings

Press **F4** or **S** to alternate between various sharpness modes. The sharpness/contrast settings should be set separately for the **Local (CCTV) Mode** and the **Formatted Text Mode**.

Changing the Text/background Color

In order to switch between available colors, press **F3** or **C**. Continue to press **C** until you have chosen the desired color.

In **Local (CCTV) mode**, the available color options are:
- black-on-white,

- inverted (white-on-black),
- color

In **Formatted Text Mode**, the available color options are:

- black-on-white,
- inverted (white-on-black),
- green-on-black,
- yellow-on-black,
- yellow-on-blue
- white-on-blue

Rotating the View of an Object

To rotate a document in the **Local (CCTV) Mode**, 90 degrees at a time, press F6 or R on your keyboard. The preview window allows you to navigate your document, and will assist you in starting to read the magnified text. You do not have to rotate a document to read it. Orientation of the document on the screen does not matter when the document is being read.

Using the Preview Window

There are three ways of viewing a page in **CCTV Mode**. The first is a standard magnified mode.

The second is a magnified mode with a little picture-in-picture window on the right top corner of the screen. The picture-in-picture window displays the image of the whole page. A focusing rectangle in the preview window identifies an area of the page which is currently displayed on the screen.

The third mode is a full screen preview mode, where the preview of the whole page is displayed on the screen.


Full screen preview or picture-in-picture preview may be separately disabled in the Preview Window tab of the **Settings** dialog. Click **Application** on the toolbar and go down to **Settings**. The Settings dialog will appear. On the Settings dialog, select the **“Preview Window”** tab. In the **“Show Preview Window”** area, select the preview window type that you want to enable (you can select either or both).

When camera is “on” only **Full Screen Preview** can be displayed, unless **“Enable real time zoom”** checkbox in the **Preview** tab of the **Settings** dialog is checked.

To show/hide the preview window or alternate between small and full screen preview windows see below:

By keyboard: Press **F5** or the **V** key.

The image below shows the full screen preview window enabled; the red focus rectangle is positioned in the center. You can move the red focus rectangle around, to wherever you want to start reading. Once you have moved the red focus rectangle to the desired location, you can either dismiss the preview window (**F5**), or go directly to **Formatted Mode (F2)**. The displayed magnified text will start from the portion of the document highlighted by the red focus rectangle. The focus rectangle disappears. The thickness and color of the focus rectangle can be changed via settings dialog.

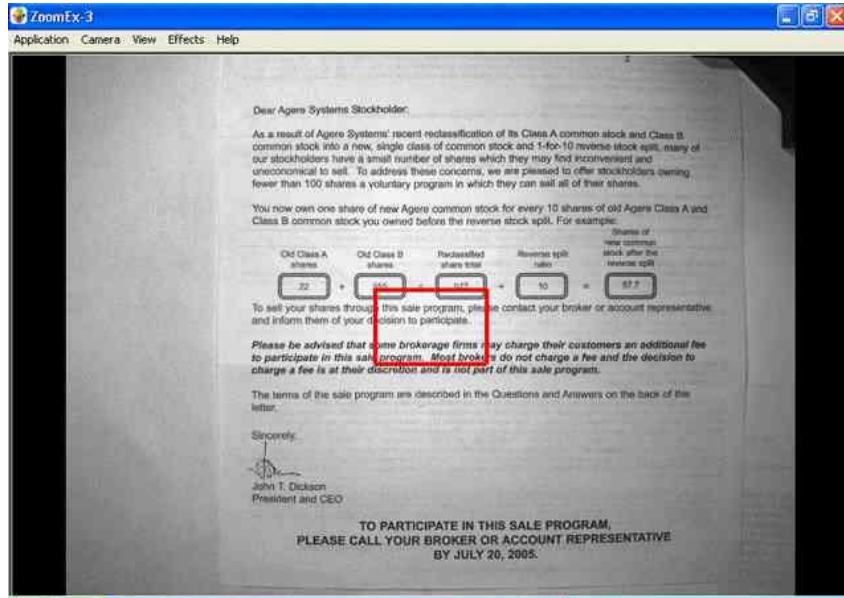

To move the red focus rectangle around the document:

By keyboard: Use the keyboard arrows to move the rectangle around. (Make sure X-Y table is checked under the **general** tab in **settings**.)

By mouse: Left-click the document, to have the red focus rectangle jump there.

You can customize how the focus rectangle will look. On the main menu, click **Application** and go down to **Settings**. The Settings dialog will appear. On the Settings dialog, select the "**Preview Window**" tab. You can select focus rectangle color (red, blue or green) and border (thin, medium, thick). Changes made to the preview window do not take effect until you click "OK".

Changing the Advanced Display Characteristics

Fine-tuning the Contrast Settings

To change the contrast:

Press the Ctrl key and the ">" on the keyboard in order to increase contrast; press Ctrl key and "<" on the keyboard in order to decrease contrast.

Note: make sure Ctrl key is pressed prior to "<" or ">" key. Pressing "<" or ">" keys without pressing the Ctrl key will adjust camera brightness.

Changing the Brightness Settings

By default, the camera operates in auto-brightness mode. In order to overwrite auto-brightness mode and change the brightness: press '<' on the keyboard to decrease brightness; press '>' to increase brightness.

Note: Brightness can be changed only while camera is 'on', in **Local (CCTV) Mode**. When in **Formatted Text Mode**, first switch to **Local (CCTV) Mode** – press space bar, which will re-start the camera.

Creating and Managing Books

A Book is an electronic copy of a book that you can create using the Zoom-Ex camera.

Zoom-Ex allows you to scan and save multi-page documents to a computer. You can view saved books at any computer where a Zoom-Ex device may not be present but Zoom-Office software is installed. While reading a Book you can use all the same document browsing, magnification, and formatting capabilities that are available for viewing documents with Zoom-Ex.

You can also listen while the book is read to you page by page. You can save the whole book as text and use any third party software to read the text file.


A Book allows you to bring your book almost virtually anywhere with you. Just save it to a CD or upload it to your MP3 player.

Zoom-Office can be used separately from the Zoom-Ex device, to read saved documents.


Creating a Book

Have a book or a multi-page document handy for scanning and reading with Zoom-Ex. Place your document under the Zoom-Ex device.


Find the Book menu and click on it or click **(Alt + B)** and then the **Enter** key.

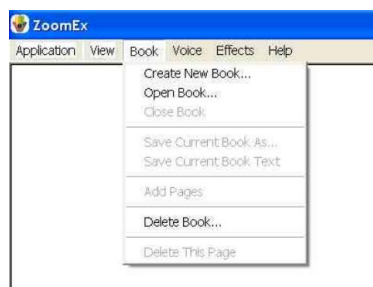

The **Create Book** dialog will appear. Put in the book title and the author's name, and click **Create**.

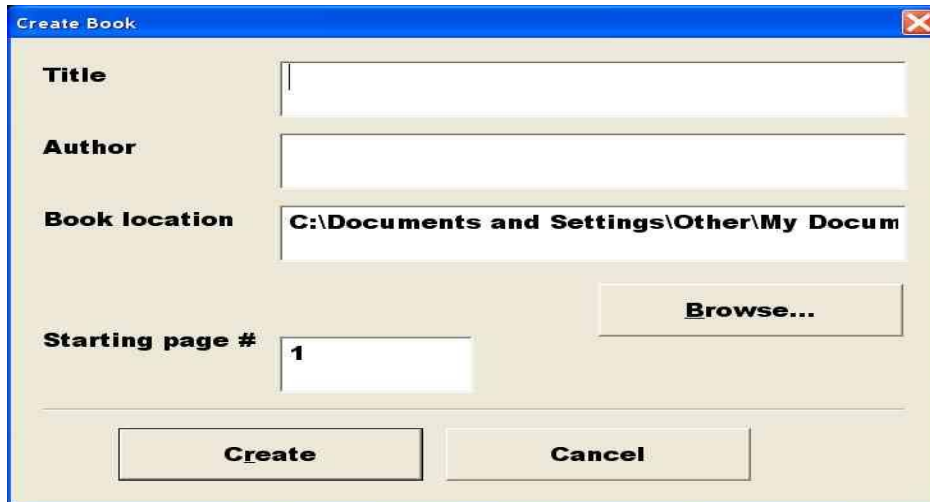

The dialog will go away and the **Add Pages** bar will appear instead.

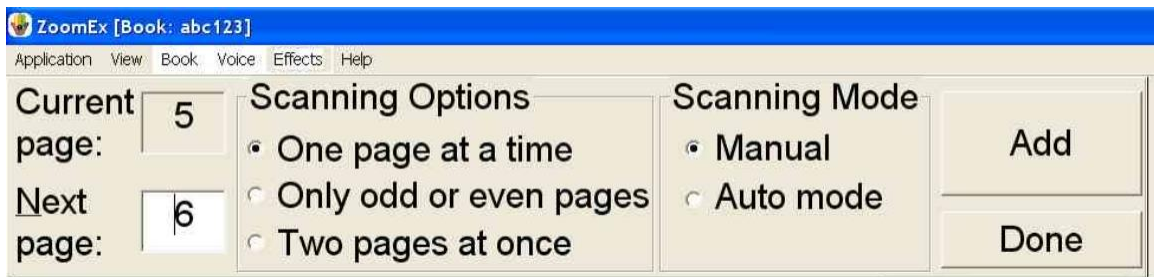

You can save any kind of book using Zoom-Ex; small paperback books, large books with page sizes up to 8.5'x11', and magazines.

Choose How to Scan a Book

Start by selecting one of the three Scanning Options:

1. Scan one page at a time.

Use this option to scan a small amount of large pages.

2. Scan only odd or only even pages.

Use this option to scan a large amount of pages in a large book. Open the book and place the left or right page under the camera. If a portion of the second page also appears under the camera, Zoom-Ex will ignore it. Do not move the book until you have scanned all the odd pages you want.

Once you finished scanning odd pages, do the same for the even pages:

Start by turning the book 180 degrees. Be sure that the even pages are now under the camera. The fact that the pages appear upside-down does not matter. Zoom-Ex rotates the pages for you automatically when you want to read the book.

Change the **Next page number** to the first even page number. Start scanning the even pages.

When you finished scanning even pages, click the **Done** button. A Book will automatically sort and keep all the pages in the right order.

3. Scan two pages at once.

Use this option to scan small sized books. If you place the open book under the camera and can see both pages in the preview window (F5), you can save both pages at once. While using this option, the book orientation is important. Always put the book in the landscape format and align the edges of the book with the Zoom-Ex base.

When you scan two pages at once, both pages will be saved with their respective numbers. You will be able to view each page separately.

If the Book fails to separate two pages, this will be indicated by an error sound and will be saved as one page. In this case you can delete that page (See "Deleting a Page in a Book" section), adjust the book and save the same pages again.

Note: The default selection is to scan one page at a time.

Scanning Modes

There are 2 types of scanning modes:

1. Manual Mode

In this mode, you press the **A** key or click the **Add** button to scan and save the page. You will hear a shutter sound indicating that the page has been scanned and saved.

2. Auto Mode

In this mode, click the **Start Auto Mode** button to start scanning and saving pages. Then simply turn the pages. The Zoom-Ex camera has motion detectors so it will scan each page as soon as the page image is still for a few seconds. It waits while you turn the pages. **Auto Mode** allows you to use both hands to hold the pages in place, since you don't have to operate the mouse or keyboard to push the **Add** button.

Note: The default selection is **Manual**.

Adding Pages to a New Book

Now that you've chosen how to scan and save a book, let's start adding pages.

1. Manual Mode:

- a) Put the first page of your document under the Zoom-Ex camera. It should appear on the screen.
- b) Click the **Add (A)** button. Use your hands to keep the book flat. In a second you will


hear a shutter sound. The page is now saved, and the **Current page** field should display **1**, while the **Next page number** should display **2** if you are saving one page at a time or **3** if you are saving two pages at a time, or even (odd) pages.

- c) Put the next page of your document under the camera and click the **Add** button or press **A** on the keyboard.
- d) Save as many pages as you need by repeating the previous step.
- e) If you are saving many pages, you can always take a break and continue saving pages later.

2. Auto Mode:

- a) Click the **Start Auto Mode** button.
- b) Place the open book under the camera; use both hands to flatten the page. In a few seconds you will hear the shutter sound, which lets you know the page has been scanned and saved.
- c) Turn the page, again keep the page flat and in a few seconds you will hear the same sound.
- d) Continue turning pages until you have all the pages you want scanned and saved.
- e) When you are done or need a break, click the **Stop Auto Mode** button and remove the book from under the camera.
- f) When you are completely finished saving pages, click the **Done** button. The **Add Pages** book bar will disappear, and the **Read Mode** book bar will appear.

Note: You can add more pages to this book later. See “Adding Additional Pages to an Existing Book” section of the manual for more details.

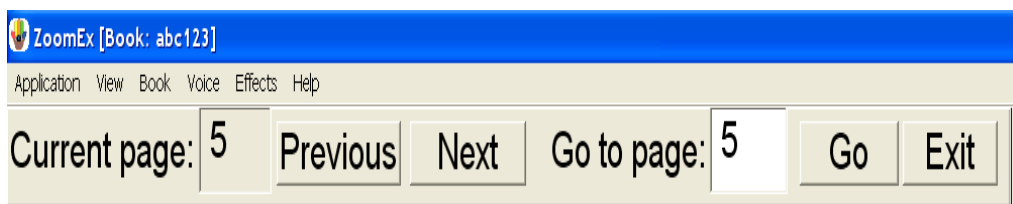

The **Read Mode** bar allows you to easily read your saved book with navigation buttons. For more information on reading your book with the **Read Mode** book bar, see Reading a Saved Book topic below.

If you are done with your book for now, click the **Exit** button to close the book.

More Tips on Saving a Book

1. Place a piece of white paper under the camera and place a book on the paper. Then Zoom-Ex will not scan the background as part of your document.
2. Make the book as flat as possible. If you are saving a thick book with a lot of pages choose the **Auto Mode** and use both hands to flatten the book.
3. When you are saving a book in **Auto Mode**, Zoom-Ex will automatically adjust brightness to the light conditions at that time. If you are taking a lot of time saving all of the pages, take a break occasionally. Click **Stop Auto Mode** and then click **Start Auto Mode** to allow Zoom-Ex to readjust brightness according to the illumination of your room as it may have changed.
4. When you are saving a lot of pages in **Auto Mode**, you may notice you have to wait longer to hear the shutter. It could be due to one of three reasons:
 - a) The camera detects some motion such as wind, unstable hands, etc.;
 - b) The camera missed a motion when you flip the page; or
 - c) Zoom-Ex turned the camera off because the conditions have changed; for example, the lights in the room were turned on or off.

You can always restart the camera and click the **Start Auto Mode** button again.

5. If you are using **Manual Mode** be sure that the page is stable on the screen before clicking **Add (A)**

Reading a Saved Book

To enable the continuous reading of the saved book go to Application/ Settings/and select. Book tab. Next, select the check box near the Continuous Book read mode. Then press the Apply button and OK button. This option allows you to read a book from page one to the last page unstopped.

To browse through the saved pages of your document:


1. Go to the **Book** menu and select **Open Book**.
2. The **Open Book** dialog will show the books in the default folder **My Documents\Zoom-Office\Books**. If you want to open the book that was saved in another folder, use the **Browse** button to select the desired folder. The location of the selected folder will be displayed in the Book location field. The list of books in this folder will be displayed on the screen.

Note: Make sure you select the folder that contains all of your books, and **not** the folder that bears the name of a specific book. In that folder you might see the folder which bears the name of the specific book. Do **not** select it.

3. Select the book you would like to view, and click the **Open** button.

Note: The **Open** button is disabled if no book is selected; as soon as you click any book in the list, the **Open** button will become enabled and clickable.

The **Open Book** dialog box disappears. The **Read Mode** bar appears at the top of the screen; the first page of the chosen document is displayed on screen as well.

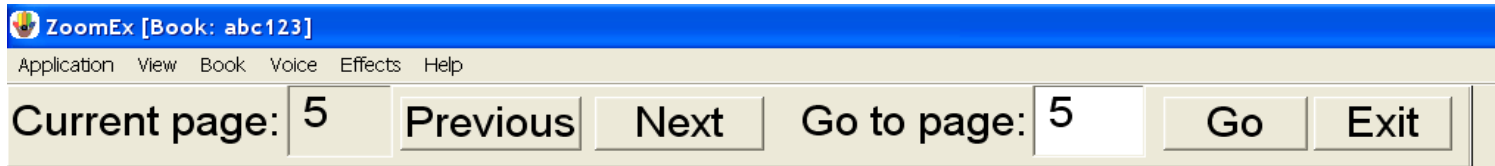

The **Read Mode** bar has the **Previous** and **Next** buttons, along with the **Go to page** field and the **Go** button to jump to a page out of order. The **Read Mode** bar also has the **Current Page** field, which displays the number of the page currently being viewed.

If the **Status Parameter** box is checked under **General Settings**, Zoom-Ex will tell you the current page number.

1. You can magnify the image of the current page, format the text part of the image, and use all other functions available for viewing documents with Zoom-Ex to make your reading efficient and more enjoyable. Zoom-Ex will remember all of your selections when you move from page to page or from book to book. If you use the Save only odd or only even pages option while saving the book, you will have to rotate only the first even (odd) page. Zoom-Ex will rotate the rest of the pages automatically.
2. Click the **Next** button or letter **N** on the keyboard to go to the next page of your document or click the **Previous** button or the letter **P** on the keyboard to go to the previous page. You can also enter a page number, and click the **Go** button to jump to that page.
3. When you are done browsing the book, click the **Exit** button.

Note: When the cursor is on the "**Current Page**" or "**Go to Page**" field and you are moving from page to page using Previous (**P**) or Next (**N**) buttons, you will hear **N or P** instead of the "Next" or "Previous". To avoid this use TAB to move the cursor.

Managing Books

Adding Additional Pages to an Existing Book


To add additional pages to an existing book:

1. Open the book by going to the **Book** menu and go down to **Open Book**.
2. Once the **Read Mode** bar appears, go to the **Book** menu and select **Add Pages**
3. The **Add Pages** Book bar will appear.
4. Place the page you want to scan under the camera, set its page number, and press the **Add** button or the letter **A**. You can also use the **Auto Save Mode** and/or save two pages at a time. See "Creating and Saving a Book" section for details.
5. Repeat step 4 as needed.
6. When you finish adding new pages, click the **Done** button. If you are saving pages in **Auto Mode**, click the **Stop Auto Mode** button first and then click the **Done** button. The **Add Pages** book bar will disappear, and the **Read Mode** book bar will appear again.


Deleting Pages in an Existing Book

In order to delete a page in a book:

1. Open the book by going to the **Book** menu, and click on **Open Book**. The **Read Mode** Book bar will appear.
2. Navigate to the page that you want to delete by putting the page number into the **Go to page** field and pressing **Go**.
3. Go to the **Book** menu, and click on **Delete This Page**.

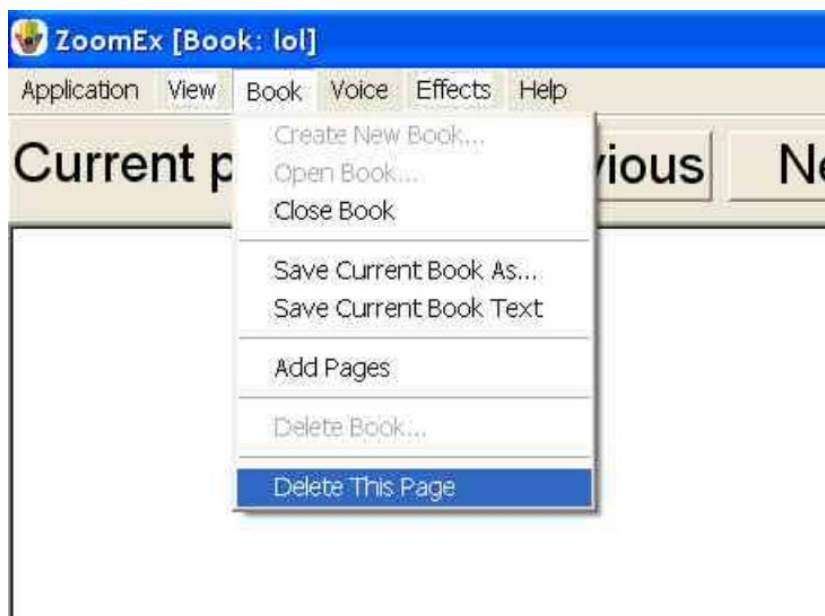


4. You will be asked to confirm your instruction. Click yes.

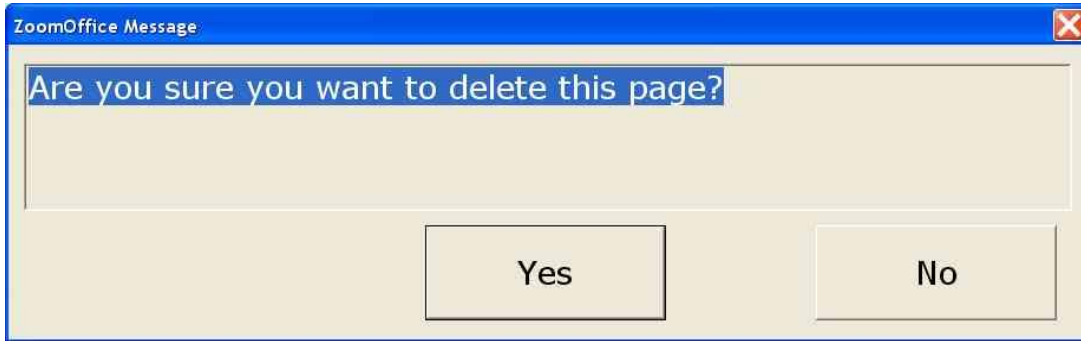

5. The page will be deleted. The display will automatically move to the next page or to the previous page if the deleted page was the last page of the document.

Deleting a Book


1. Go to the Book menu, and click on Delete Book.

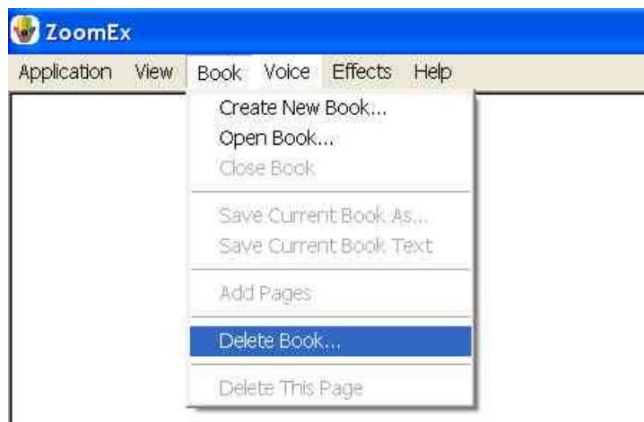

2. The Delete Book dialog box will appear. Select the book that you wish to delete and click the **Delete** button. You cannot delete an open book.




3. You will be asked to confirm your instruction. Click yes.

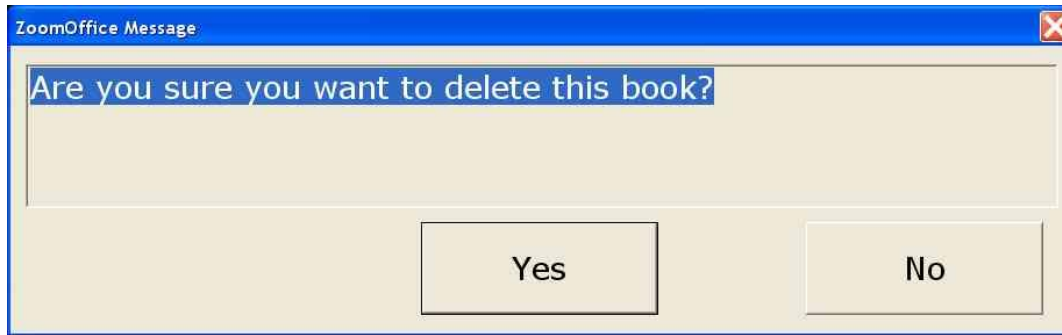

Closing a Book

To close a book you can click the **Exit** button or go to the **Book** menu, and click on **Close Book**.

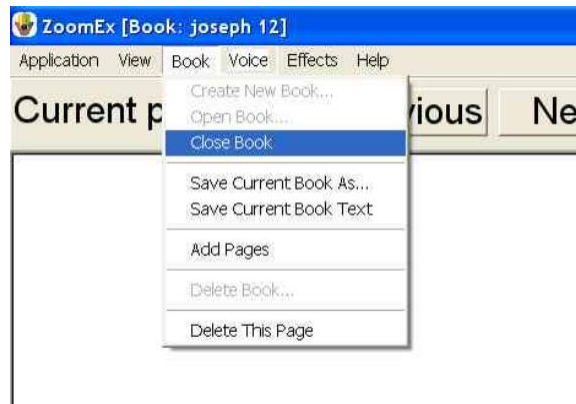

After you have scanned a book, you can convert the whole book to one text file. Under the **Book Menu** click on **Save Current Book as Text**. The file is saved in the folder **My Documents\ZoomOffice\Texts**. Depending on the power of your computer, it could take approximately from 3 to 6 seconds to save each page. For 200 pages the process may take from 10 to 20 minutes.

If the Status box is checked under **General Settings**, Zoom-Ex will tell you the current page number.

Saving the entire Book as MP3 audio file

After you have scanned a book, you can convert the whole book to an audio MP3 file. Under the **Book Menu** click on **Save Current Book as MP3**. The file is saved in the folder **My Documents\ZoomOffice\MP3**.

Please note that MP3 file cannot be opened from the Zoom-Ex application. You can open it from the directory on your computer where it has been saved.

Copying a Saved Book

The fastest way to initially save a book is to save it on the hard drive. After you have scanned your book, you can use the **Save Current Book As** menu item to copy the book onto a CD or a flash drive.

1. Go to the **Book** menu, and click on **Save Current Book As**.
2. Use the **Browse** button to select the drive and/or folder where you want to copy the book.
3. Enter the **New Name** for the book.
4. Click the **Save** button.

It may take a few seconds to save the book.

Useful Tips for the Zoom-Ex Camera

If you are working with more than one windows application at a time, Zoom-Ex window may lose focus. You will know that it has, if you have tried to turn the camera on or off by pressing the space bar and nothing happens. Click the mouse anywhere on the screen to bring focus to the Zoom-Ex application.

There is a check box under the **General tab** of the **Settings** window: **“Use arrow keys to simulate X-Y table”**. The default is unchecked. The unchecked option allows you to use arrow keys while reading the text. For example: to move to the next word, sentence, etc. If you noticed that arrow keys do not behave properly for reading, check this setting.

Zoom-Ex Quick Reference Guide (5.0)

Notebook (Laptop) Keystrokes for Reading Functions

To Scan and Start reading: E

Stop Zoom-Ex Camera: Space

Pause/Resume reading: X

Switch between Virtual Magnification and Formatted Modes: (F2) or (F)

Switch between Live Magnification and Scanning Mode: Y

Note: When in Zoom-Ex to switch back from Magnification (Live CCTV) Mode to Scanning Mode, press (Y) first, then the space bar to start the camera.

Reading by word:

Next: Control + Right

Previous: Control + Left

Reading by character:

Next: Right

Previous: Left

Reading by sentence:

Next: Down

Previous: Up

Reading by section:

Next: Control+Down

Previous: Control+Up

Speech Output Control Keystrokes

Increase voice speed: (Shift + Q) or (F9)

Decrease voice speed: (Shift + Z) or (F8)

Hear page description: (Shift + E).

Hear current control: Shift

Stop speaking current control: Control

Zoom-Book Keyboard Command Keys

Add page: A (While creating a new book).

Next page: N

Previous page: P

Zoom-Book Menu Accelerator Keys

(Alt + B) is an accelerator for Book Sub-Menu.

Key	Function
B	Book Menu, Create new book
O	Open Book
D	Delete Book
L	Close book
V	Save current book as
A	Save current book as text
G	Add pages
T	Delete this page

Zoom-Ex Visual Keyboard Command Keys

Key	Alt Key	Function
+ / -		Zoom In/Out
F2	F	Switch between Preview/Magnification and Formatted Modes
Page Up/Down		Go one screen Up/Down in Formatted Mode
F3	C	Toggle color
F4	S	Toggle sharpness
F5	V	Show/Hide preview panel in Magnified Mode
F6	R	Rotate image 90 degrees
CTRL + SHIFT + S		Quick Image Saved
CTRL+L		Turn LED On / Off
Home		Go to the first line of the page in Formatted Mode
CTRL + S.		Save current image

If option **“Use arrow keys to simulate X-Y table”** is checked, the following keys may be used:

Arrow keys		In CCTV Mode to move magnified portion of the image Up, Down, Left, Right. In Formatted Mode go one line Up/Down.
< / >	, / .	Brightness adjustment when camera is On

When Running Line Auto Scrolling Mode is Enabled

< / >	, / .	Speed adjustment
-------	-------	------------------