


ONYX[®] HD Remote Control Quick Reference Card

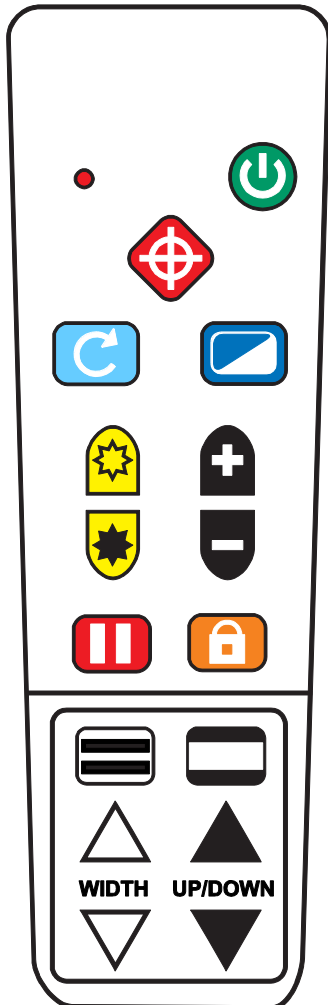

LED (Red) Flashes when a button is pressed



Power (Green) Turns unit on and off



Find (Red) Displays crosshairs and zooms out to help aim the camera



Flip (Light Blue) Cycles through four different picture orientations



Color Mode (Blue) Cycles through different color modes



Brightness (Yellow) Increases and decreases screen brightness



Zoom (Black) Zooms camera view in and out



Freeze Frame (Red) Freezes the current image displayed on the screen



Focus Lock (Orange) Turns Auto Focus on and off



Reading Lines (White and Black) Displays two horizontal or vertical reading lines



Mask (Black and White) Displays shades or masks to cover parts of the screen



Width (White) Increases and decreases space between shades, masks, and reading lines



Up and Down (Black) Moves shades, masks, and reading lines on the screen