#### **MAGic Most Used General Commands**

Description	MAGic Keyboard	Keyboard Combination
Launch MAGic: Starts MAGic or closes MAGic if it is running.	MAGic <sup>®</sup>	CTRL+ALT+M (only starts MAGic)
<b>Toolbar:</b> Displays or hides the MAGic toolbar (user interface).	×	CAPS LOCK+U
MAGic Context Menu: Opens the MAGic context menu for quick changes to MAGic settings.		CAPS LOCK+APPLICATION KEY or CAPS LOCK+RIGHT MOUSE BUTTON CLICK
Decrease Rate: Decreases the MAGic speech rate.	*	
Increase Rate: Increases the MAGic speech rate.	7	

440557-001 Rev. F



# **MAGic Magnification Commands**

Description		MAGic Keyboard	Keyboard Combination
Visual Enhancements: Toggles all visual enhancements including magnification, mouse enhancements, color enhancements, focus enhancements, and cursor enhancements.		M	CAPS LOCK+DELETE
Mag Level Toggle: Switches between the current magnification level and 1x magnification. All visual enhancements will remain active.		A A	CAPS LOCK+SHIFT+DELETE or CAPS LOCK+MOUSE WHEEL CLICK
Increase Magnification	<b>4</b>	<b>4</b>	CAPS LOCK+NUM PAD PLUS or CAPS LOCK+MOUSE WHEEL UP or CTRL+EQUALS
Decrease Magnification		<b>!</b>	CAPS LOCK+NUM PAD PLUS or CAPS LOCK+MOUSE WHEEL UP or CTRL+DASH



### **MAGic Visual Enhancement Toggles**

Description	MAGic Keyboard	Keyboard Combination
Mouse Enhancements: When enabled, mouse enhancements make the mouse pointer easier to track and maneuver on the screen.	K	CAPS LOCK+F4
Cursor Enhancements: When enabled, this makes the cursor easier to locate and follow when writing or editing text.	ab	CAPS LOCK+F8
Color Enhancements: Color enhancements can be adjusted to meet individual needs for handling glare and contrast.	<b>M</b>	CAPS LOCK+F12
<b>Focus Enhancement:</b> Displays a rectangular border that identifies and tracks the control that currently has the focus.	:=	CAPS LOCK+F
<b>Text Viewer:</b> Turns the Text Viewer on or off. When enabled, the Text Viewer window displays across the screen and simultaneously shows the same text that appears on the magnified screen.	A LA	CAPS LOCK+CTRL+V



#### **MAGic Most Used Special Application Commands**

Description	MAGic Keyboard	Keyboard Combination
Research It: Quickly finds information on the Web without using a Web browser to navigate through complex pages. Research It uses lookup sources to find information, such as the meaning of a word, the local weather forecast, top news stories, and more.	<b>?</b> (1)	Press and release INSERT+SPACEBAR, and then press R or INSERT+WINDOWS Key+R (using default lookup source)
Links List: Opens a dialog box containing a list of links in the current Web page or Word document.		INSERT+F7
Select and Say: Reserved for a future MAGic feature.		

### **MAGic User Assigned Commands**

Description	MAGic Keyboard	Keyboard Combination
M1, M2, and M3: These keys are reserved for users to assign to their preferred MAGic functions, shortcuts, or Windows commands using the MAGic Keyboard Manager.	M1, M2, M3	User assigned key combinations



# **MAGic Reading Commands 1 of 2**

Description	Keyboard Combination
Say All	INSERT+DOWN ARROW or CAPS LOCK+A or CAPS LOCK+LEFT MOUSE BUTTON CLICK
Say Currently Selected Text	INSERT+SHIFT+DOWN ARROW
Say Contents of Clipboard	CAPS LOCK+X
Say Current Character	NUM PAD 5
Say Current Word	INSERT+NUM PAD 5
Spell Current Word	INSERT+NUM PAD 5 (twice)
Say Next Word	INSERT+RIGHT ARROW
Say Prior Word	INSERT+LEFT ARROW
Say Current Line	INSERT+UP ARROW



# **MAGic Reading Commands 2 of 2**

Description	Keyboard Combination	
Say to Cursor	INSERT+HOME	
Say from Cursor	INSERT+PAGE UP	
Say Current Sentence	CAPS LOCK+S	
Say Next Sentence	CAPS LOCK+CTRL+S	
Say Prior Sentence	CAPS LOCK+ALT+S	
Say Current Paragraph	CAPS LOCK+P	
Say Next Paragraph	CAPS LOCK+CONTROL+P	
Say Prior Paragraph	CAPS LOCK+ALT+P	
<b>Skim Reading:</b> Use it to quickly browse through long documents by reading the first part (or the first sentence) of each paragraph.	INSERT+CTRL+DOWN ARROW	



### **MAGic Speech Toggles**

Description	MAGic Keyboard	Keyboard Combination
Mouse Echo: Turns mouse echo on or off.		CAPS LOCK+M
Typing Echo: Toggles the typing echo on and off.		
<b>Speech Toggle:</b> Toggles between having speech on all the time (full speech) versus having limited speech (Speech On Demand).	3))	CAPS LOCK+F3



#### **MAGic Dual Monitor Commands**

Description	MAGic Keyboard	Keyboard Combination
Jump to Next Display: When using dual monitors, this key allows users to quickly move the computer focus and the mouse pointer to the other monitor.	ŢŢ	CAPS LOCK +SHIFT+D
View Lock: When using dual monitors, press and hold while moving the mouse pointer across the screen or to another monitor without scrolling the display.	모	Hold CAPS LOCK+SHIFT and move mouse between displays
Move Current Application to Other Display: Moves the current application or window in focus to the monitor on the right or the left.		WINDOWS KEY+SHIFT+RIGHT ARROW or WINDOWS KEY+SHIFT+LEFT ARROW

