

<b>Key commands for reading functions</b>	
Start or Stop Camera	Spacebar
Scan and Start Reading	E
Pause or Resume Reading	X
Read by Character	Next: Left Arrow Previous: Right Arrow
Read by Word	Next: Control + Right Arrow Previous: Control + Left Arrow
Read by Sentence	Next: Down Arrow Previous: Up Arrow
Read by Paragraph	Next: Control + Down Arrow Previous: Control + Up Arrow

<b>Speech output key commands</b>	
Increase reading speed	Shift + Q or F9
Decrease voice speed	Shift + Z or F8
Hear page description	Shift + E
Stop speaking	Control
<b>While a camera is on, you can:</b>	
Turn LED On or Off	Control + L
Adjust brightness	Period or Comma
Cancel processing	Escape

<b>Book key commands</b>	
Create New Book	Alt + B
Open a Book	Alt + B + O
<b>While a book is open:</b>	
Add page	A
Next page	N
Previous page	P