


TOPAZ[®] XL Control Panel Reference Card

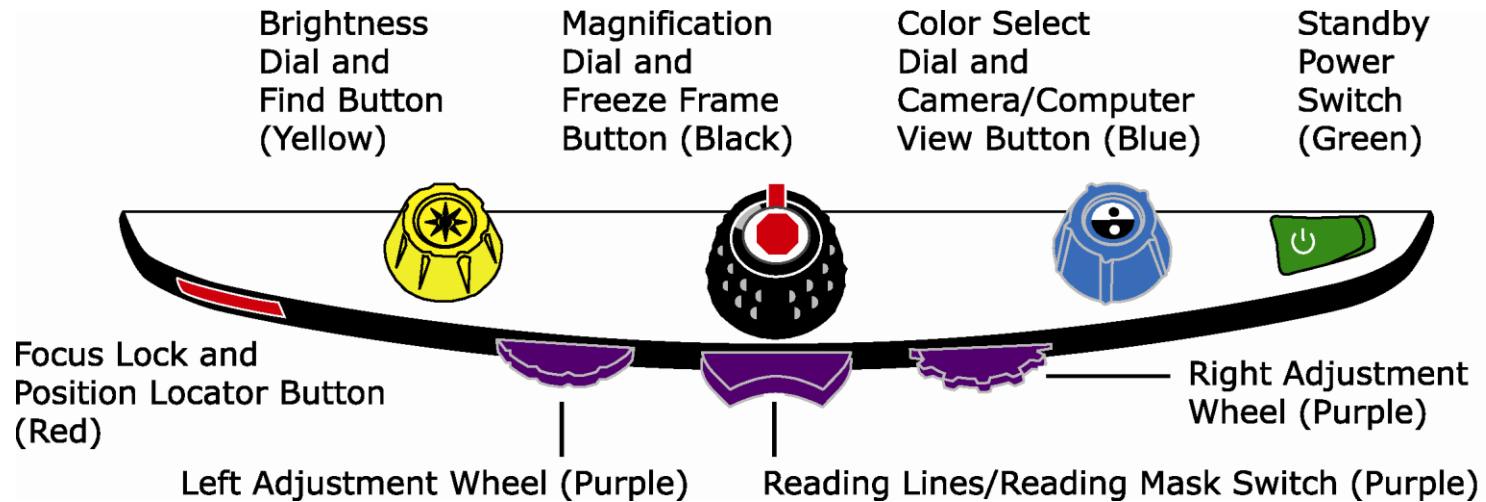

Dial / Button / Switch	Functionality
Brightness Dial / Find Button (Yellow)	<ul style="list-style-type: none"> • Turn to increase or decrease screen brightness. • Press and hold center to zoom out using the Find feature. Release to zoom back in.
Magnification Dial / Freeze Frame Button (Black)	<ul style="list-style-type: none"> • Turn to increase or decrease magnification. • Press center to switch Freeze Frame on or off.
Color Select Dial / Camera and Computer View Button (Blue)	<ul style="list-style-type: none"> • Turn to choose a color mode that best suits your vision needs. • Press center to switch between the TOPAZ XL camera and an attached computer. • Press center to switch the single horizontal reading line on or off if not connected to a computer.

Dial / Button / Switch	Functionality
Standby Power Switch (Green)	<ul style="list-style-type: none"> • Press to the right to place the TOPAZ XL in standby mode. Press to the left to turn on TOPAZ XL from standby mode.
Focus Lock Button / Position Locator Button (Red)	<ul style="list-style-type: none"> • Press to switch Focus Lock on. Press a second time to switch the Position Locator beam on. Press a third time to turn off both the Position Locator beam and Focus Lock mode. • Press and hold simultaneously with the center of the yellow Brightness dial to switch LED reading lights on and off.
Reading Lines and Reading Masks Switch (Purple)	<p>Switch is located underneath the plastic cover (not shown) below the control panel.</p> <ul style="list-style-type: none"> • Press to the right to cycle through horizontal lines, vertical lines, vertical masks, horizontal masks, and none. • Press to the left to cycle through horizontal masks, vertical masks, vertical lines, horizontal lines, and none.
Left Adjustment Wheel (Purple)	<p>Wheel is located underneath the plastic cover (not shown) below the control panel.</p> <ul style="list-style-type: none"> • Turn to the left or right to move both reading lines or reading masks up and down or left and right on the screen.
Right Adjustment Wheel (Purple)	<p>Wheel is located underneath the plastic cover (not shown) below the control panel.</p> <ul style="list-style-type: none"> • For horizontal lines or masks, turn the wheel to move the top line or mask up or down. This increases or decreases the distance between horizontal lines or masks. • For vertical lines or masks, turn the wheel to move the right line or mask to the left or right. This increases or decreases the distance between vertical lines or masks.